Baby Care

Art.No.1022061

## BABY NASAL ASPIRATOR

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**Features:**

This nasal aspirator is an effective & safe product that uses your own suction power to effectively remove mucus from the nose of the baby, thereby providing immediate relief to the baby's nasal congestion. More efficient as suction generated is more powerful than the standard bulb nasal aspirator or the battery operated nasal aspirator. Scientifically designed – soft – “nose tip” and “mouth piece pipe” for safe and efficient removal of mucus.

**Please read the instructions carefully and keep the packaging for future reference. Care & use:** Before use, wash your hands. Place the – “mouth piece pipe” in your mouth and gently place the tip end of the nasal aspirator into one of your baby's nostrils. Gently suck from the – “mouth piece pipe” without removing the tip end of the nasal aspirator from the baby's nostrils. The mucus is drawn up and collected into the aspirator. After collecting mucus, open up the tip and carefully dispose off the mucus hygienically from the aspirator.

**To Clean:** Before first use, wash all parts of the nasal aspirator with warm water and a mild detergent, rinse thoroughly with clean water and dry. Always wash before and after each use. Dispose off the mucus, hygienically from the aspirator and wash under running tap water. Then make a mixture of warm water and a mild detergent. Open the nose tip and place all the parts of the nasal aspirator into the mixture. Wash thoroughly and repeatedly in this mixture of warm water and mild detergent until the nasal aspirator is clean. Rinse thoroughly with clean water and dry. Do not put in boiling water as this can damage the product. Do not use abrasive pads or cleaners. Do not clean or allow to come into contact with solvents or harsh chemicals. This could result in damage. Do not leave the product in direct sunlight or near heat because this can lead to damage. **For your child's safety and health: WARNING! This product is NOT a toy.** Never leave a child unattended while using this product. This product should only be used by an adult. Always use this product with adult supervision. Keep all components not in use out of children's reach. Keep out of reach of children. Throw away if there are any signs of damage or wear.

- Soft and safe
- Effective
- Suitable for dual age group (0-3m; 3+m)
- Easy and fast

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